


LEMEX

CLEANER/REODORANT

AUSTRALIAN MEDICAL DEVICE CLASS 1

ARTG No. 125529

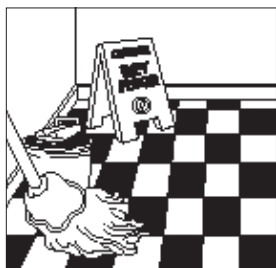

BENEFITS:

- Inexpensive, general purpose detergent
- Fresh long lasting perfume
- pH Neutral - Excellent Materials Compatibility
- Non-corrosive
- Class 1 Medical Device

LEMEX is an inexpensive lemon perfumed liquid suitable for general use as a cleaner or reodorant. LEMEX is suitable for surface cleaning and use on other Class 1 Medical Devices and has been entered onto the Australian Register of Therapeutic Goods (ARTG). Using LEMEX as directed ensures Healthcare facilities comply with current Healthcare regulations.

LEMEX is a neutral detergent containing a powerful cationic surfactant and a fresh long-lasting lemon perfume. Being neutral it is non-corrosive to most fibres and common hard surfaces including normal domestic, commercial and most hospital/medical flooring.

The principle use of LEMEX is as a mopping detergent. LEMEX can also be used throughout Hospital Operating Theatres including for cleaning of operating tables, theatre trolleys, lights and other equipment in the operating theatre and environs. VIRACLEAN should be used where general disinfection is required. Where used for cleaning Medical Devices LEMEX should be used as a cleaner only and subsequent disinfection, where required, should be done with an Instrument Disinfectant such as INSTRUMAX PINK.

Normal use instructions are as follows:

	DILUTION IN POTABLE WATER
GENERAL CLEANING	
Normal Cleaning:	25mL/L with potable water
Heavy Cleaning:	50mL/L with potable water
Deodorising:	200mL/L with potable water
CLEANING MEDICAL DEVICES	
No visible soiling:	25mL/L with potable water
Soiled with blood or body fluids	50mL/L with potable water

*Make Fresh Solutions Daily.

Lemex - September 2015 1

CHEMICAL COMPOSITION AND PROPERTIES:

Appearance:.....	Clear yellow liquid
Colour (concentrate):.....	Yellow liquid
Colour (dilute solutions):.....	Light yellow liquid
Composition:.....	Surfactant blend
Detergent System:.....	Cationic
Odour:.....	Lemon
pH (concentrate):.....	6-8 (Neutral)
Solubility:.....	Soluble
Rinsibility:.....	Total
Abrasives:.....	Nil
Hazardous Status:.....	Non Hazardous
Volatility:.....	Nil
Flammability:.....	Nil
Dangerous Goods Rating:.....	Nil
Shelf life:.....	2 years

SAFETY DATA: Information is listed in the Material Safety Data Sheet. Refer to the medical section of the Whiteley Corporation website: www.whiteley.com.au .

PACKSIZE: 2 x 5 Litre
15 Litre

ENVIRONMENTAL STATEMENT:

Whiteley Corporation manufactures products that comply with Australian and International environmental policy. Whiteley Corporation aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley Corporation products are supplied in 100% recyclable containers. Whiteley Corporation and its staff are fully committed to ensuring that our environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

© This document remains property of Whiteley Corporation Pty Ltd.
Alterations are not permitted without prior written authorisation from Whiteley Corporation Pty Ltd.

Lemex - September 2015 2